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VERSION 05

**SIDLY**

SIDLY Sp. z o.o.  
Chmielna 2/31  
00-020 Warszawa  
Poland

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Authors: SiDLY Sp. z o.o.

Graphic design: SiDLY Sp. z o.o.

Operating instructions recommended for distribution among the customers of the device manufacturer who bought the device model covered by this manual. Devices purchased after July 1, 2018 from the manufacturer are subject to remote software upgrade.

The content complies with the requirements of national and EU legislation in force in November of 2015.

Instructions apply effective law on copyright and related rights.

Any changes made to the manual without the knowledge and permission of the author's exempt him from responsibility of disputes resulting from these changes.

All rights to reprint or copy are reserved - only allowed with the permission of the manufacturer.

Any changes made to the manual without the knowledge and permission of the SiDLY's exempt him from responsibility of disputes resulting from these changes

Instruction is available at [www.sidly-care.eu](http://www.sidly-care.eu)

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The screenshot shows the SIDLYCARE web interface. At the top, there are navigation tabs: 'Opiekunowie' and 'Użytkownicy'. Below that, a sub-menu includes 'Ogólne', 'Leki' (selected), 'Alerty', and 'Raporty'. A user profile card for 'Kowalczyk Wanda' is visible, dated 'Wtorek, 12 styczeń 2018, godzina 18:35'. The main content area is titled 'Leki' and contains a table of medications. A green 'Dodaj lek' button is in the top right of this section. The table has columns for 'Nazwa leku', 'Data przyjęcia', and 'Opis'. Each row includes 'Edytuj' and 'Usuń' buttons.

Nazwa leku	Data przyjęcia	Opis
Aspirin C	Dzisiaj, 20:00	On the other hand, we denounce with righteous indignation
Witamina B	Dzisiaj, 20:00	-
Activlab CLA Green Tea Plus	08:00, 13/01/18	On the other hand, we denounce

Rys. 7. Menu systemu [www.sidly-platform.com](http://www.sidly-platform.com) – przypomnienie o lekach

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**CONTENTS**

---

1.....	<i>GENERAL</i> .....	31
1.1.	<b>INTRODUCTION</b>	31
1.2.	<b>THE DEVICE IDENTIFICATION</b>	32
2.....	<i>SAFETY OF USE AND WARNINGS</i> .....	33
2.1.	<b>GENERAL SAFETY RULES</b>	33
2.2.	<b>SAFETY USAGE RULES</b>	34
2.3.	<b>USING THE CONNECTORS, PORTS AND BUTTONS</b>	35
2.4.	<b>WORKING TEMPERATURE</b>	35
2.5.	<b>SAFETY SIGNS</b>	35
2.6.	<b>RESIDUAL RISK DESCRIPTION AND ASSESSMENT</b>	35
3.....	<i>SIDLY CARE DESCRIPTION</i> .....	36
3.1.	<b>INTENDED USE</b>	36
3.2.	<b>TECHNICAL CHARACTERISTICS AND CONSTRUCTION</b>	37
4.....	<i>PRELIMINARY OPERATIONS AND PUTTING INTO SERVICE</i> .....	38
4.1.	<b>DEVICE DELIVERY</b>	38
5.....	<i>USE OF DEVICE</i> .....	39
5.1.	<b>GENERAL PRINCIPLES</b>	39
5.2.	<b>INITIAL START</b>	39
5.3.	<b>SWITCHING ON AND OFF</b>	40
5.4.	<b>DEVICE SYNCHRONIZATION</b>	41
5.5.	<b>THE DEVICE FUNCTIONALITY</b>	43
5.5.1.	<b>BAND REMOVAL SENSOR</b> .....	44
5.5.2.	<b>THE MEASUREMENT OF VITAL PARAMETERS</b> .....	44
5.5.3.	<b>DETERMINING A USER'S GEOLOCATION</b> .....	46
5.5.4.	<b>MEDICINES REMINDER</b> .....	46
5.5.5.	<b>SOS BUTTON / PANIC BUTTON</b> .....	47
5.5.6.	<b>FALL DETECTOR</b> .....	48
5.5.7.	<b>GPRS COMMUNICATION</b> .....	48
5.5.8.	<b>PEDOMETER</b> .....	49
5.5.9.	<b>BAROMETER</b> .....	49
5.6.	<b>BATTERY CHARGING</b>	49
6.....	<i>MAINTENANCE</i> .....	51
6.1.	<b>CLEANING</b>	51
6.2.	<b>STORAGE</b>	51
7.....	<i>WITHDRAWAL</i> .....	52
8.....	<i>TESTS AND CERTIFICATES</i> .....	52
9.....	<i>WARRANTY</i> .....	53

## 1. GENERAL

---

*Thank You for Your trust and purchase of SiDLY. We hope that thanks to it You provide a feeling of security for yourself and your closes, irrespective of the distance.*

### 1.1. Introduction

**WARNING!**

*This operating instructions should be attached to SiDLY Care device.*

This operating instructions is a basic equipment of the device. Be familiar with its contents prior to device use. It is recommended to keep the operating instructions for future use.

Please read all of these instructions before using the device for a first time. Reading this entire manual and proceeding carefully with the principles pre-

sented here ensures the safe usage and minimizes the risk of accidents or malfunctions.

These instructions are valid only for the device, which has been delivered with. It is identified with the device according to its type and serial number inscribed on the title page.

**WARNING!**

*Before starting work, make sure you read this entire manual. If any information given in these instructions are incomprehensible please contact for help directly to the manufacturer (business address and contact information are located on the title page of this manual and on the product label).*

The SiDLY Care device has been designed and manufactured in accordance with safety requirements of Council Directive 93/42/EEC of 14<sup>th</sup> June 1993 concerning medical devices, but inappropriately utilized may cause hazards to the user health and life or third parties. Therefore, utilizing the device you must follow the provisions of the manual. Only persons who are familiar with the device and its operating instructions can use it.

The SiDLY product must be used only for its intended purpose. The unauthorized usage against the principles set out in this manual will void your warranty. In the event of damage in this respect, the

manufacturer may refuse to replace the device to a new one.

The manufacturer provides a complete SiDLY Care device with a dedicated charger, a tool for switching on and operation manual.

Making any modifications to the device release the manufacturer from the consequences of their implementation and may void Your warranty. Details of the warranty and service are stated in the warranty card.

In case of doubt regarding the operation of the device, observed malfunctions or defects, please contact the manufacturer at the address:

SiDLY Sp. z o.o.  
2/31 St. Chmielna  
00-020 Warsaw, PL  
tel.: + 48 667 870 126  
e-mail: [office@sidly.eu](mailto:office@sidly.eu)

**WARNING!**

*Any other way of the product use, not described in this operating instructions you must first consult the device manufacturer.*

**WARNING!**

*Information important for the user is indicated in such a way!*

**NOTE!**

This sign warns of the potential hazard in relation to the device operator or bystanders.

Not following the recommendation marked by this sign can cause accidents. Strictly follow the recommendations specified by this sign.

**WARNING!**

1. *The manufacturer reserves the right to make changes to the device and its software in order to improve it. These changes will also be introduced on an ongoing basis to the instructions. This does not mean, however, that will be introduced also for devices previously provided to the users. The latest version of the instructions for the device is available on the website [www.sidly-platform.com](http://www.sidly-platform.com)*

2. *In case of sale or rental the device to another user, always give him the operating instructions. In the event of its loss or damage, please contact your dealer to purchase a new copy.*

3. *The manufacturer is not responsible for accidents resulting from not following the safety usage rules published in this manual and the wider use of the device.*

4. *The device is not serviceable. In the case of any defect manufacturer exchange the device to a new one.*

**1.2. The device identification**

---

Identification data of the device are provided on the label, which is located on the back of SiDLY Care.

On the label is also:

- Name and address of the manufacturer,
- Serial number,
- Year of manufacture,
- CE marking


## 2. SAFETY OF USE AND WARNINGS

---

### 2.1. General safety rules

When using the device You are required to respect the general health and safety regulations and the rules laid down for medical devices and consumer electronics devices. In addition to the above You are required to respect the following guidelines, in order to eliminate possible occurrence of hazards:

- the device should be used only in accordance with the intended purpose,
- do not make any modifications in the device construction,
- charge built-in battery only the supplied charger that is included as a part of the device kit.

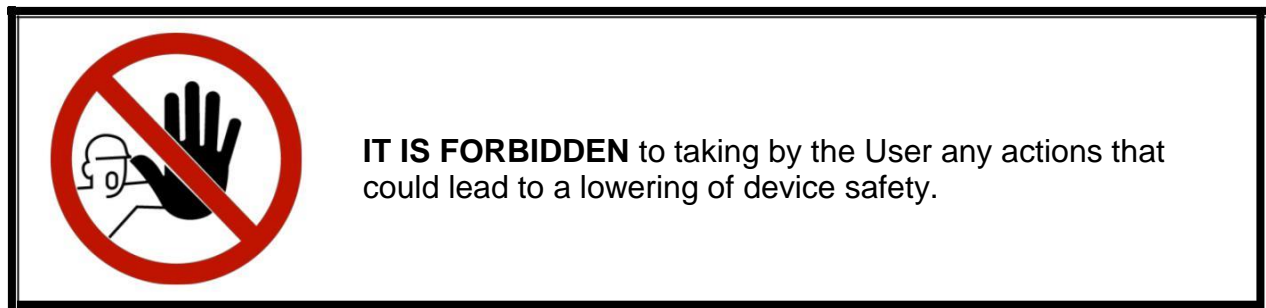Use the SiDLY Care device only for the intended purposes and in a manner described by the manufacturer.

Before use, please read these instructions and observe the following rules regarding the elimination of hazards and taking appropriate precautions.

- Use the device only in the intended manner (see ch. 3.1).
- It is forbidden to use the device by children under 3 years old.
- It is forbidden to use SiDLY Care for purposes other than those specified in this manual.
- Do not make unauthorized changes to the SiDLY Care construction.
- Charge the device only a charger supplied with SiDLY Care.

## 2.2. Safety usage rules

---

**NOTE!**

The device must strictly adhere to the user's wrist to work properly. We recommend You to place precisely the device on his left wrist, correctly and in a way that not create a discomfort feeling.

**WARNING!**

*Do not wear the device for bathing or swimming in the pool.*

**WARNING!**

*Direct strong impact in the band will damage it.  
A puncture of the battery by a sharp tool may cause fire due to the release of energy stored in the lithium-polymer battery.*

**WARNING!**

*Remove the device before you will be subjected to MRI (magnetic resonance imaging) or X-rays.*

- It is forbidden to use the device with a soiled, scratched or damaged sensors, that are located on the underside of the device.
- It is forbidden to use devices that have become wet or frosted.
- It is unacceptable to use the device after You find any damages or improper operation. You should contact with the manufacturer at first.
- It is unacceptable to charge the battery when the device is placed on sb's wrist. Before starting the charging process, remove the device from his hand.
- Nano-SIM card is placed in the device by the manufacturer.
- The device is not intended to perform its functions on the plane because of the lack of possibility of sending SMS messages to the server.
- The device is sensitive to strong impact therefore it is unacceptable to intentionally throwing band.
- We recommend removing the device before you will be subjected to MRI (magnetic resonance imaging – the risk of band damaging) or X-rays (the risk of electronic damaging).

### 2.3. Using the connectors, ports and buttons

---

Never use too much force when pressing a connector to USB port, or use excessive force to press the button with the switch.

Such action may cause damage that is not covered by the warranty.

If there is not easy to connect plug and USB port, they probably do not match.

Check whether there is something between the USB and USB socket that can block the correct connection.

Make sure that the plug fits into the socket and that the plug is properly positioned in relation to the socket.

### 2.4. Working temperature

---

The device has been designed for work and storage in normal conditions at ambient temperature  $-5^{\circ}\text{C} - +45^{\circ}\text{C}$ .

If SiDLY Care is used or stored at temperatures outside this range it may be damaged.

Do not expose the device to sudden temperature or humidity changes.

During use or charging the battery, SiDLY Care may become hot, which is a normal phenomenon.

Do not leave the device for a long time in heated or cooled-site (eg. in a car or under direct sunlight).

### 2.5. Safety signs

---

Pictogram (safety sign) on the device informs about an unacceptable method of disposal.

This sign should always be clear and clean, unobstructed and visible.

It is forbidden to dispose the device to containers of municipal waste.

Placed on the label.

### 2.6. Residual risk description and assessment

---

The device is designed and constructed in accordance with the current state of the art and the applicable standards.

Despite the efforts of the manufacturer to ensure the safety and elimination of hazards when using the SiDLY Care,

some elements of risk during use cannot be avoided.

They usually result from the improper method of usage. The residual risk may arise in emergency situations, arising in particular from disregarding with the operating instructions or from not paying proper attention during user-to-device interaction.

The highest hazard occurs when conducting the following, prohibited activi-

ties described in section. 2.1 and 2.2 and:

- using the device in a manner not as intended purpose.
- using the device by children under three years old,
- exposing the unit to extreme temperatures.

It is possible to limit the residual risk to minimum provided that the following recommendations are fulfilled:

- carefully reading and following the operating instructions,
- fastening the device to accurately clung to the wrist.
- no sudden movements during measurement,
- control the level of battery charge.

### 3. SiDLY Care DESCRIPTION

---

#### 3.1. Intended use

*Fig. 1. SiDLY Care – overview*

The device belongs to medical products of IIa class. It is the active non-invasive device for monitoring heart rate and skin temperature.

The device should be used by people familiar with its intended use, and its technical characteristics.

The intended use of the device is using it for the purpose of monitoring the patient's condition. It is dedicated to working with your mobile phone thanks to nano-SIM card installed by manufacturer. The card number is placed on the product label that is located on its bottom.

We also recommend to type this number on the cover page of this manual.

The device is reusable until its functionality will be maintained. Is adapted to:

- the pulse rate and skin temperature measuring,
- in addition to:
- to designate geolocation coordinates,
  - signal the need for taking medications
  - send alarm information onto the declared phone number,
  - detect the user's falls if he or she stays down for more than 120 seconds after falling.

**Unauthorised changes to the device without the manufacturer permission exempt him from liability for damages.**

Meeting the requirements regarding the use of the device in accordance with the manufacturer's instructions and

strict compliance with the recommendations is a condition of intended use. Using the device for other purposes will be considered as improper use.

In particular, it is unacceptable to use of the device by children under 3 years old.

### 3.2. Technical characteristics and construction

Table 1

Parameter	Unit	SiDLY Care
Weight	g	45
Overall dimensions:		
width	mm	max 44
height	mm	max 224
depth	mm	max 15
adjustment range (on the wrist)	mm	173 - 248
Minimum GSM signal strength required for communication	dBm	-108
The range of signal strength for proper operation	dBm	-95
The pulse measurement range	bpm	0 - 250
User skin temperature measurement range	°C	30-45
Uncertainty of user skin temperature measurement	°C	±0.525
Battery charging time	h	2 - 3
Temperature of the device storage and use	°C	od -5 do +45
Life cycle	years	1
Water resistance	IP	54
Charging voltage	V	5
Maximum charging current	mA	1000
Charging socket	-	mikro USB

Table 2

Parameter	Unit	Charger for SiDLY Care
Charger	typ	GSM0811
mains supply	V (AC) / Hz	100-240 / 50-60
output voltage	V (DC)	5
device protection class	-	II

The device design is made of medical elastomer and polypropylene (which has been examined in terms of allergies, irritation and cytotoxicity).


The device has a form of wristband with adjustable belt fastening on the wrist.

The SOS button described also in Braille is placed at the front shield. The 3 LED warning are placed on the button, each in a different color.

There is a protective cap of micro USB connector on the side and switch operated with a special tool (supplied with the unit).

The device label and sensors used for performing the diagnostic measurements are placed on the back of the cover (fig. 3).


The device is also equipped with a sensor that detects the band taking off.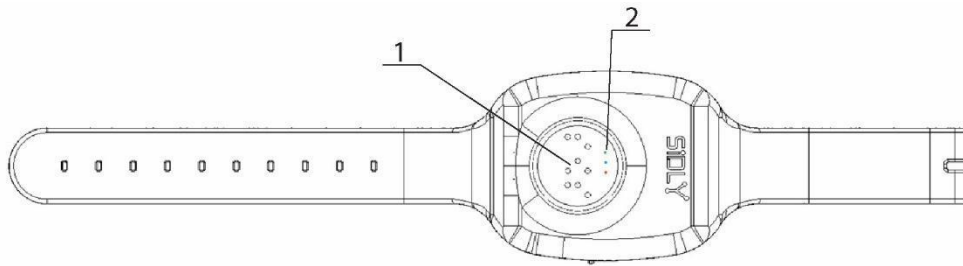

1 – SOS button labeled in Braille


2 – LED warning

*Fig. 2. SiDLY Care – rear view*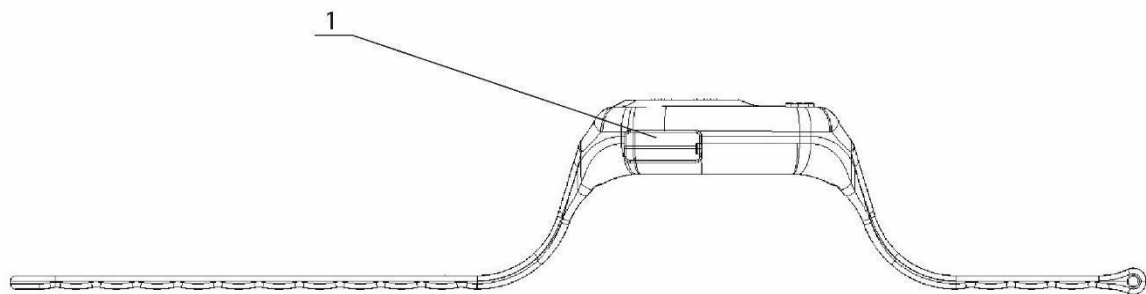

1 – protective cap of micro USB connector and switch

*Fig. 3. SiDLY Care – side view*

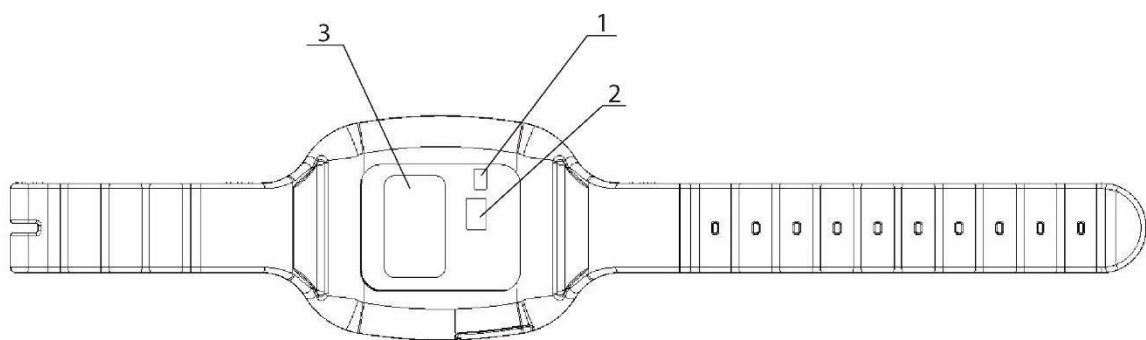1, 2 – sensors

3 – device label

*Fig. 4. SiDLY Care – rear view*

## 4. PRELIMINARY OPERATIONS AND PUTTING INTO SERVICE

### 4.1. Device delivery

The device is supplied as a ready-to-use product.

It is packaged in a cardboard box, which in addition to the devices are:

– dedicated charger,

– pin for switching the device on / off,

- operating instructions,
- warranty

Read these instructions before you use the device.

## 5. USE OF DEVICE

---

### 5.1. General principles

---

Before each use, check the technical condition of the device.

In case of deformation, cracks or other mechanical damage such a device should be withdrawn from use.

The device is designed for wearing on the wrist (it is recommended to place it

on the left wrist) in a manner such as on the figure below (fig. 5).

It should be placed on the wrist after wearing on clothes. Putting on clothes in a situation when the band is placed on wrist may damage the device - clothes may catch device, damaging them.

*Fig. 5. SiDLY Care – placed on left wrist*

### 5.2. Initial start

---

During the initial activation procedure You must connect the device to the charger at first and carry out battery formation. It's done by fully device

charging. An estimated time to charge your device is 2 hours. Full charge will be signaled by red diode going off.

#### **WARNING!**


*SIM card installation must be done on the off device!*

1. To mount the nano-SIM card, remove the plastic cover of the device with silicone band. Place the nano-SIM card in the slot on the side edge.
2. Unpack the nano-SIM card purchased from your telecom operator.

Rys. 6 Removed device from the silicone housing

3. Hold the SiDLY Care with the SOS button up and the sensors on the left side next to the microUSB input insert have a SIM card slot.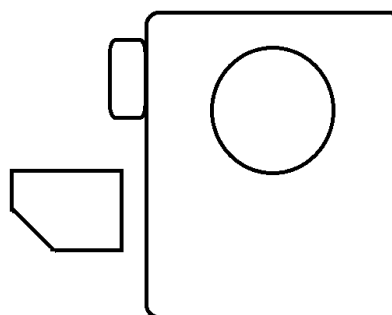

Rys. 7 Way rack-mountable nano-SIM card into the device

4. Insert the SIM card into the device so that the bevelled corner is on the left side of the SIM card
  5. Carefully and gently push the card in the slot.
- If any resistance occurs when inserting the card into the unit, immediately stop the operation and start again. Inserting the card into the slot can damage the card and the SiDLY Care device!!!

**WARNING!**  
*The installed card in the device have to be without a security PIN!  
 APN card must be internet!  
 Nano-SIM card must work in 2G network!*

You can change the APN name. To change the APN name, please follow the instructions below:

1. Download the nRF UART 2.0 application from the Google Play store.

2. Turn on the SiDLY Care device.

3. Wait until the blue LED on the device switch off.
4. Launch the nRF UART application on your mobile phone.
5. Click Connect in the nRF UART application.
6. A list of devices appears, select SiDLY Care from the list. Tap &Set apn: new\_apn
7. Press ENTER and then SEND

### 5.3. Switching On and Off

**WARNING!**


*Nano-SIM card is placed in the device at the production stage, you do not have the opportunity to exchange the card.*

**WARNING!**

*There is a special pin supplied in the kit to switch on / off the device.*

Turning on the device is done by connecting the band to loading or by ON / OFF switch located under the USB plug. To switch device on or off:

- Press the switch ( fig. 9) using a special pin provided with SiDLY Care.
- Raise the cap placed on the side of the device (fig. 8),

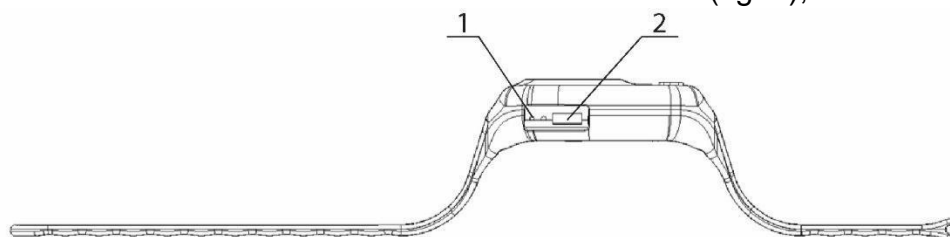1 – Switching On and Off

2 – micro USB connector

*Fig. 8. SiDLY Care – side view*

*Fig. 9. Special pin to switch SiDLY Care on / off*

**WARNING!**

*There is a special pin supplied in the kit to switch on / off the device.*

### 5.4. Device synchronization

In order to synchronize the device with an application should be made the user registration on [www.sidly-platform.com](http://www.sidly-platform.com) or other website dedicated by the manufacturer. During the registration process, follow the instructions. In the next step you need to download and

install SiDLY Care application from the Google Play™ store or App Store®. During the installation procedure, follow the instructions that appear on the screen (of mobile device) of the phone or tablet.

After a successful installation, add a band to the application.

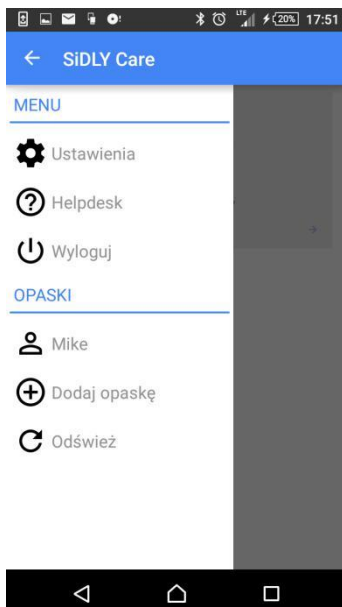
Fig. 9. Application menu for SiDLY Care on a phone or tablet

To do this, enter your band user name, the device's IMEI number available on the product label placed at the bottom

of the device or on the package label and enter at least one emergency telephone number.

**WARNING!**  
*It is required to enter at least one emergency telephone number. Entering of other two is an option.*

**WARNING!**  
*On phone number guardian listed as the emergency number are sent only SMS messages informing about the falls of user and SOS button is pressed. Notification of well or poorly founded band, battery status, and information about the results of the measurements are sent to the application "SiDLY Care"*

Fig. 10. Application menu for SiDLY Care on a phone or tablet – entering of emergency numbers

**WARNING!**  
 You have to enter the IMEI number correctly. To change the IMEI number You should delete the band from application and add it again.

After a maximum of 30 minutes from carrying the above steps, you will receive a message SMS on emergency phone number entered in application. A message is as follows, „Congratulations! Your SiDLY Care has been configured and is ready for use! Correct skin

temperature will be displayed 30 minutes after fitting the band. Please remember that skin temperature is lower than body temperature. Measurements submitted are indicative. We wish you satisfaction with using the device”. The above procedure occurs only when the device is switched on for the first time.


**5.5. The device functionality**

There are active the device functionality when you first switch the device on:

- band removal detection,
- pulse and skin temperature measurement,
- medicines reminder,
- SOS function,
- GSM / GPRS Communication,
- fall detector.

There is required a cooperation with a dedicated mobile application for proper operation of all device functionality.

The following figure shows information about the LED colours and their reaction depending on the functions carried out by the device.

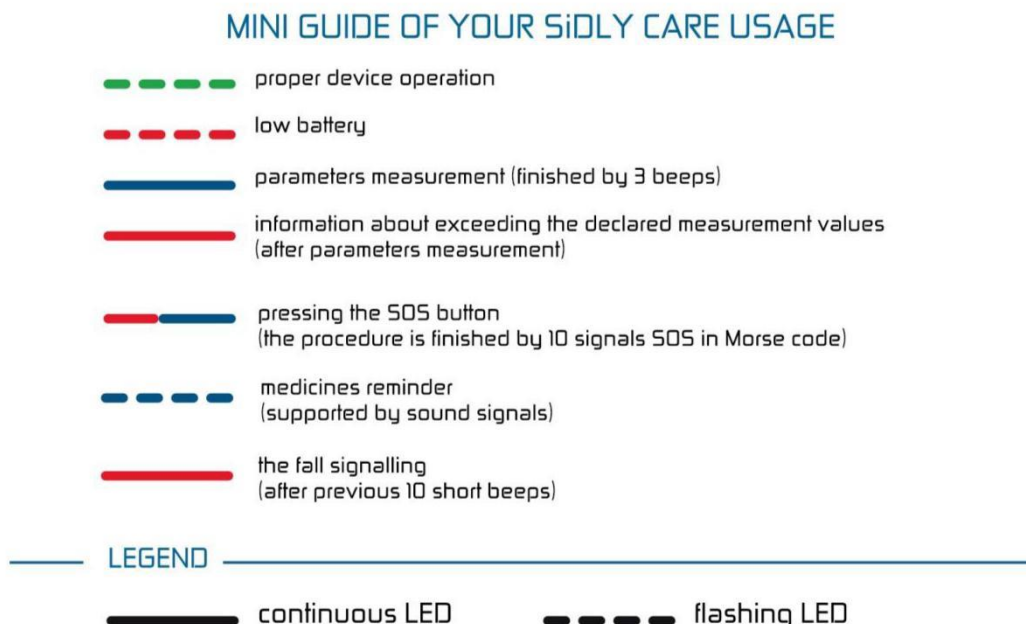*Fig. 11. Mini guide of Your SiDLY Care usage*

### 5.5.1. Band removal sensor

---

If the wristband is not wear on the wrist, the device generates a short one second beep. The sensor of the band checks the current state of the band in a few minutes. The launching or snapshot status is updated on the platform and in the mobile app at approximately five-minute intervals. If within a period of about one hour, the

wristband will not change from the assumed position on or removed from the assumed position, the message will be sent to the web platform and the SiDLY Care mobile application. If the band is set up from the hour, a message will be sent: "The band [User] worn from the hour". If the band is removed from the hour, a message will be sent: "The band from [User] removed from the hour".

### 5.5.2. The measurement of vital parameters

---

The device measures the pulse and skin temperature. Individually, skin temperature is lower than the body temperature by approx. 3-5 °C.

The measurement procedure is signalled by continuous blue LED sustained for the entire measurement duration.

The measurement can be initiated automatically by setting a time interval, describing at what time to be measured. The time interval may be occasionally changed (one-time change) caused by lack of GSM communication.

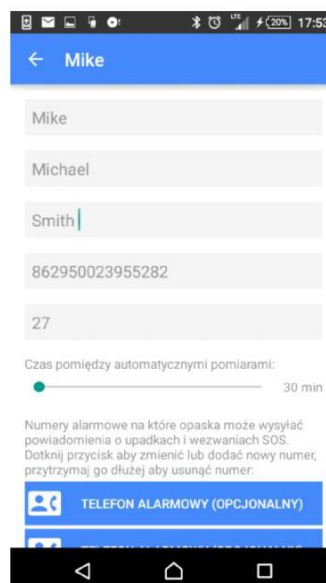
Fig. 12. Application menu for SiDLY Care on a phone or tablet – time between measurements

When the measuring procedure is finished the measured data is sending through GPRS mobile application.

Fig. 13. Application menu for SiDLY Care on a phone or tablet – measurement

The measurement results are stored in the mobile application and on the website [www.sidly-platform.com](http://www.sidly-platform.com) (or on other application dedicated by the

manufacturer), which user is logged on. The data are presented in the form of graphs and figures.

Fig. 13. Application menu for SiDLY Care on a phone or tablet – measuring pulse and temperature graphs

It is recommended that during measurement process the band must be tighten to the wrist.

user during the measurement will perform

sudden movements, the device may carry out incorrect measurement or may signal to repeat the measure-

In the case where the band will loose on wrist during measurement or the

ment procedure by three one-second beeps. Failure to follow the manufac-

turer's instructions may result in incorrect measurements.

If the measurement exceeds the declared value, the red LED is on for a time min 10 sec.

In order to carry out correct measurements, you should keep a complete cleanness of sensors placed at the bottom of the device.

In order to ensure the highest possible degree of measurements accuracy we

recommend to carry out measurements after 30 min since the user worn a band. It applies also in a case of high change of ambient temperature.

Strong electromagnetic interference generated by eg. microwaves, antennas - may cause inaccurate device readings. Carrying out measurements in the case when user is close to a source of intense heat / cold may also cause inaccurate device readings.

In no case, this product does not replace a consultation with a doctor or other medical professional. The results of measurements are only of informative type. A doctor should be contacted if any health problems are suspected. Under no circumstances the prescribed medicines should be changed without consulting a doctor or other healthcare professional.

---

### 5.5.3. Determining a user's geolocation

---

Using A-GPS the device is able to locate the user's band.

Locating the user is performed when you press the SOS button and when it detects a falling.

The location of the user is transmitted by a link to [openstreetmap.org](https://openstreetmap.org) in an SMS message sent on the declared emergency number, and presented in a mobile application.

Location accuracy resulting from used technology is up to 200 m. The manufacturer is not liable for exceeding the discrepancies localization. Location discrepancy may be result of insufficient level of telecommunication infrastructure expanding of mobile network operator.

In the case of lack of the band connection to the GPRS network location determining may not occur.

---

### 5.5.4. Medicines reminder

---

The device signals time for taking medicines declared by the user. User declares hours of taking medicines in the mobile application.

Along with the time declared enter the name of medicine or medicines that you need to take at certain times.

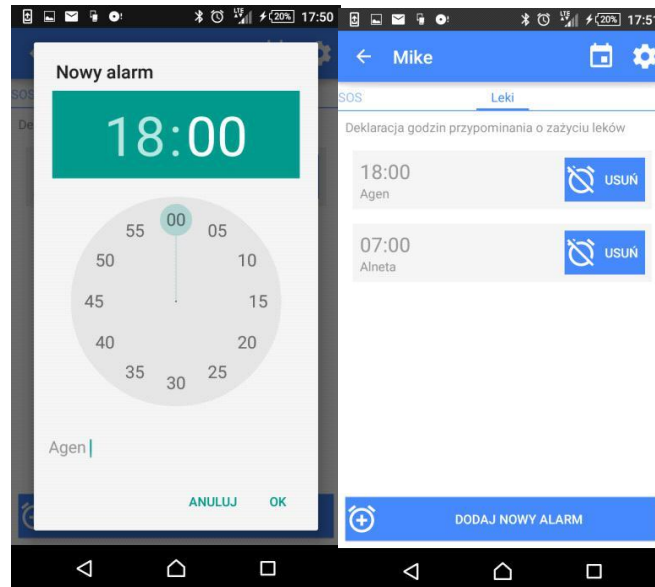
Fig. 14. Application menu for SiDLY Care on a phone or tablet – medicines reminder

The device signals time for taking medicines using beeps and at the same time light signals - flashing blue LED.

The user can declare up to 12 different alarm times to remind of medicines.

### 5.5.5. SOS button / Panic Button

Calling SOS Alarm is done by holding down the SOS button. Start of the alarm procedure signals red LED. Then the band gets the location data and after about 1 - 2 minutes sends an SMS to the declared numbers ICE (In Case of Emergency) containing header SOS !!! and the location of the band.  
[SOS !!!  
<http://www.openstreetmap.org/#map=18/52.217892/21.015528>].

After it send an SMS successful there is triggered pulse and temperature measure. It is indicated by a blue LED.

Throughout the measurement duration the blue LED lights up continuously. After measuring the device sends a second SMS that contains information about pulse and temperature and location in which the band is.

The procedure ends with 10 beeps of SOS in Morse code. Information about triggering the SOS alarm is also sent to the application: [Pulse = 67bpm, Temperature = 37.43degC  
<http://www.openstreetmap.org/#map=18/52.217892/21.015528>].

Fig. 15. Application menu for SiDLY Care on a phone or tablet – SOS Alarm

### 5.5.6. Fall detector

The SiDLY has a fall detector built-in the device.

In the case of fall detection device signals it the 10 one-second beeps of detection.

If there was a false fall detection, the user can cancel alarm by holding down the SOS button.

In case of alarm canceling the red LED lights up continuously until the procedure ends. Then the alarm message

will be sent by SMS to a declared emergency number (ICE) FALL !!!

<http://www.openstreetmap.org/#map=18/52.217892/21.015528>.

The message of fall will be sent also to the application. There is pulse and temperature measurement procedure triggered after the message sending.

The device detects falls, where a loss of user's awareness occurs, or its immobility.

#### **WARNING!**

*SOS alarm Activation and fall detection by the device results in sending SMS messages to declared numbers ICE. Measurements initiated from application level or initiated automatically by setting the time interval are stored in the mobile application and on [www.sidly-platform.com](http://www.sidly-platform.com)*

### 5.5.7. GPRS communication

The device has the ability to connect to the Internet, making it possible to transfer measurement data from a band to the application and the ability to configure the band settings.

The band connection configuring with the application and the website is car-

ried out at the application level in the "Settings", by entering the IMEI device

number. The IMEI number is available on the product label, placed at the bottom of the device or on the packaging label.

**WARNING!**

*All settings made in the band and changes of these settings will be updated during the band connection with GPRS.*

Note: You must enter the IMEI number correctly. To change the IMEI number

you should delete the band in application and add it again.

### 5.5.8. Pedometer

The device takes the measurements of the number of steps taken in the selected day. The measurement procedure is automatically started after putting the band on the hand.

The results of the measurements are saved in the application and on the website [www.sidly-platform.com](http://www.sidly-platform.com) (or different

dedicated by the producer), on which the user is logged in, in the form of the number of taken steps and in the form of the percentage result of the realized goal. The goal is declared by the user in the mobile application or on the website [www.sidly-platform.com](http://www.sidly-platform.com). The goal is the number of steps that the user makes daily.

### 5.5.9. Barometer

The device SiDLY Care takes measurements of the atmospheric pressure. The measurement procedure is taken automatically together with the measurements of the vital signals. The results of the measurements are saved in the

application and on the website [www.sidly-platform.com](http://www.sidly-platform.com) (or different dedicated by the producer), on which the user is logged in. The results are presented in the form of the bar chart, which has the measurement of the atmospheric pressure of that day.

### 5.6. Battery charging

SiDLY Care has a built-in lithium-polymer battery and charger dedicated for this device (fig. 16).

Charger parameters for SiDLY:  
Travel Charger,  
Model: GSM0811,  
Input: AC 100 - 240 V / 50 – 60 Hz,  
Output: DC 5 V, 1000 mA.

Fig. 16. Certified charger dedicated for the device

**NOTE!**

It is forbidden to charge your SiDLY when the user has it on his hand.

### Battery charging

Connect SiDLY Care to a power source using the supplied micro-USB charger.

Rechargeable batteries have a limited number of charge cycles. Frequent charging will shorten battery life.

To charge SiDLY care:

- remove the band from the user's hand,
- put the electric plug into the socket.
- place the charger the micro-USB plug in micro-USB slot of band,
- check that the LEDs indicating the charging lit up (red and green).

To complete the charging of SiDLY Care:

- check whether the red LED doesn't lights up, this is the way to indicate the end of charge,
- unplug the micro-USB plug from micro-USB slot of band,
- unplug charger from the electricity outlet.
- check whether the green LED flashes periodically to indicate normal operation.

Interruption of device charging by unplugging it from the power will cause that the red and green LEDs go off and then the green LED will flash periodically in case of proper further work.

It is required to fully charge the battery when you switch the device a first time.

The user is informed about the charging device a message sent to the mobile applications and SiDLY Care online platform reads: "The band [user] fully charged."

The band user is informed about the necessity of charging the band when the red LED diode starts blinking (if the band is not connected to the charger). In the case of close low battery message is sent to the Care SiDLY mobile applications and web-based platform

reads: "The band [User] discharged. Connect the charger to the band. "

How to use the charger can contribute to cable cracking and breaking. Charger cable may be weakened or become breakable if it is repeatedly bent in the same place. Bend cable smoothly, without breaking it. We recommend you to check frequently the cable and connectors condition, searching for holes, cracks and other damages. You should stop using the cable if there occurs described damages.

#### **WARNING!**

*The device is recommended to charge until it sends a notification of readiness to use on the mobile applications and SiDLY Care online platform.*

*It is likely that the device switched off too early can quickly unload - it results from characteristics of lithium-polymer charging.*

## **6. MAINTENANCE**

---

### **6.1. Cleaning**

---

If SiDLY Care comes into contact with any substance likely to cause contamination (eg. dust, dirt, ink, cosmetics), the device should be immediately cleaned.

To clean SiDLY Care:

- unplug all cables and turn SiDLY Care off (press and hold the power button for about 3 seconds using pin supplied with device)
- to clean, use a soft, lint-free cloth or cleaning and disinfecting wipes,
- do not let moisture get into the inside of the device,
- do not use cleaning agents (detergents) or compressed air.

At the rear part of the devices are placed sensors, which must be cleaned carefully with a soft, dry cloth.

Cleaning agents and abrasive materials contribute to rub-off sensors and can disrupt a proper functioning of the device.

The device does not require special maintenance.

Long lasting and efficient use of the device is dependent on the skillful handling and correctness of cleaning soiled device.

Daily cleaning service is based on cleaning the device by cleaning and disinfecting tissue.

### **6.2. Storage**

---

We recommend non taking the device off for night. This will prevent the pa-

tients that they forget about again its taking on.

If, however, you have to take the device off, put it in a safe place without exposure to fall from a height or losing.

We recommend charging the device at fixed times to not forget about the charging and re-using.

It is unacceptable to charging or store the device in an ambient temperature above 60 °C (risk of explosion).

## 7. WITHDRAWAL

### **WARNING!**

*Do not dispose the device in municipal waste. It is subject to the same rules as disposal of electronic products.*

Guaranteed life cycle of the device is 12 months (see warranty conditions). This does not mean, however, that after 12 months the device is totally in-operable. With careful handling of the

device it is possible that it works longer than 12 months.

In the case of the total device consumption or damage it must be removed from service and sent for disposal.

## 8. TESTS AND CERTIFICATES

---

### **WARNING!**

*Unauthorized changes or modifications by SiDLY may violate electromagnetic compatibility (EMC) and cause the loss of the right to claim warranty. The product complies with EMC requirements, among others, in terms of cooperation with compatible devices (charger).*

The device has been tested for compliance with the requirements of Council Directive 93/42/EEC by TÜV Rheinland (Notified Body No 0197) and obtained CE certificate.

**9. WARRANTY**

---

SIDLY Sp. z o.o.  
2/31 St. Chmielna  
00-020 Warsaw, PL  
NIP:7010435677  
Regon:147363415  
KRS:0000519291

---

Stamp of seller

**WARRANTY**

SIDLY Care device

Serial number .....

Year of manufacture.....

designed to monitor important life parameters.

**Date of sale**.....

**Buyer**.....

**Period of warranty validity is 12 months from date of delivery.**

**Warranty service is performed by the producer.**

.....  
**Date**

.....  
**Signature of the seller**

**When complaints you must show warranty card and proof of purchase (invoice, receipt).**

The buyer receives the warranty from the supplier for a period of 12 months counting from the date of device delivery.

**NOTE FOR THE BUYER!**

**The purchaser should carefully read a warranty card.**

**GENERAL RULES OF WARRANTY PROCEEDINGS**

---

- 1) The warranty does not exclude, not limit or not suspend the rights of the buyer arising from the provisions of the statutory warranty for defects in the goods sold.
- 2) Warranty covers defects and damage of the device caused by the fault of the producer, due to material defects, improper producing or assembly. By provision of guarantees the manufacturer undertakes to complete device replacement for a new device free of defects.
- 3) The Guarantor declares that subject covered by this Warranty has been issued free of defects and was manufactured in accordance with current standards.
- 4) The user notifies complaint to a dealer from which the machine was purchased or to the manufacturer (address on the warranty card).
- 5) The manufacturer will consider the complaint as soon as possible and no later than within 21 calendar days from the date of delivery defective product by the consumer to the guarantor.
- 6) The user should submit a complaint at the latest within 14 days after the date of damage.
- 7) The warranty is extended by a period in which a complaint was considered, counting from a date of filing the complaint and until the day of providing a new device free of defects to the user.
- 8) The manufacturer will not accept warranty claims if there were made any technical changes in a device without the manufacturer approval, if device was improperly stored, operated or maintained. It also concerns identified prior attempts to repair the device.
- 9) The warranty does not cover:
  - a) defects caused by the machine operation not in accordance with the principles of operation, by alterations, design changes made by the user or third parties;
  - b) defects caused by user abandon of activities related to the machine maintenance, cleaning and adjustment;
  - c) defects caused by external influences and particularly mechanical damage, damage due to the action of chemicals;
  - d) natural wear and tear consistent with the properties or intended device use including;
  - e) the damages caused by mechanical, thermal, chemical, and other factors caused by user activity or by external forces;

- f) damages that occurred as a result of transport and during loading machines operated by any carrier.
- 10) The producer is not obliged to bear the cost of compensation due to production losses or any other damages resulting from damage, hidden defects or device failure.
- 11) This guarantee does not affect the rights and obligations arising from the provisions of national law.
- 12) The warranty is voided if there are made modifications, adaptations or changes in machinery construction by persons other than the manufacturer or the person designated by him.

#### ELEMENTS TO BE RETURNED TO THE MANUFACTURER.

1. Complete device with packaging, charger and proof of purchase. I

have read the warranty term and conditions

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